


mediven® elegance, mediven® plus, mediven® comfort, mediven® for men

Standard without seam							
Circumferences	I	II	III	IV	V	VI	VII
eG thigh with silicone topband	43 – 48	45 – 52	49 – 56	53 – 60	56 – 64	60 – 68	64 – 72
eG thigh with silicone topband extra wide	49 – 57	53 – 62	57 – 67	61 – 72	65 – 77	69 – 81	73 – 85
eG	43 – 57	45 – 62	49 – 67	53 – 72	56 – 77	60 – 81	64 – 85
eF	39 – 52	41 – 56	44 – 60	47 – 65	50 – 69	53 – 73	57 – 78
eE	30 – 37	33 – 40	35 – 43	37 – 45	39 – 48	41 – 51	42 – 53
eD	27 – 33	29 – 36	32 – 39	34 – 42	36 – 45	38 – 48	40 – 50
eC	28 – 34	30 – 37	33 – 40	35 – 43	37 – 46	39 – 49	41 – 51
eB ₁	23 – 27	24 – 29	26 – 32	29 – 35	31 – 37	33 – 39	35 – 41
eB	18 – 20	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30	30 – 32
eY	26 – 31	28 – 33	29 – 35	31 – 37	32 – 38	33 – 40	34 – 42
eA	17 – 22	19 – 24	21 – 26	23 – 29	25 – 32	27 – 34	28 – 35

Medi Plus Extra Wide Class II only		
	Calf Measurement	
Size 1	34 - 38cms	
Size 2	37 - 41 cms	
Size 3	40 - 44 cms	
Size 4	42 - 44 cms	
Size 5	44 - 48 cms	
Size 6	46 - 50 cms	
Size 7	48 - 52 cms	

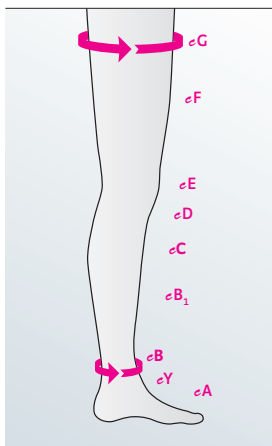

Length	
eG regular	72 – 83
eG petite	62 – 71
eD regular	38 – 43
eD petite	33 – 37

Toe	
Standard with open toe	

Instructions for fitting Medi Stockings




I feel better.

1. Since the greatest pressure on the leg is exerted at the ankle it is important to measure first the circumference of the ankle "b" Measurement "b" (narrowest point just above the ankle bone) determines the stocking size.
2. Depending on the style of stocking you should measure the circumference of the following points:

Below Knee	Thigh Length
b = ankle	b = ankle
c = calf	c = calf
	g = thigh

Circumference:
(= small letters)

Length:
(= capital letters)

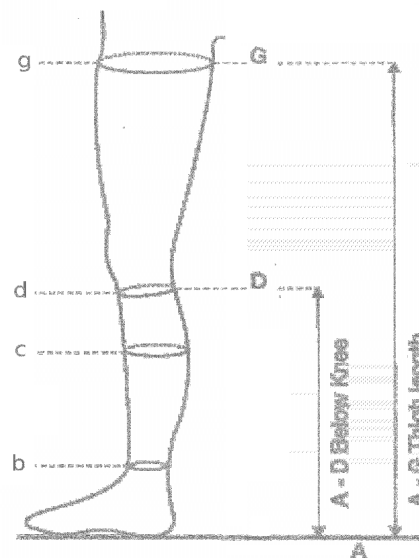

3. In order to determine the appropriate length , please measure the length from the heel (point "A") to the following point:
i.e. *calf* A to D
Thigh A to G
4. As a general rule, the circumference of both legs should be measured so that any discrepancies between the two legs can be taken into consideration. medi stockings are available in seven circumference sizes (I - VII) which fit 90% of patients. With the remaining 10% of patients medi made to measure stockings (custom fitted) are an option.
5. For further details please check the latest made to measure chart

INDICATIONS

Compression class	Recommended compression hosiery
Class 1 Mild varicosis without a significant predisposition to oedema, early varicosis in pregnancy	mediven elegance (closed toe) mediven plus (open toe)
Class 2 For more noticeable symptoms, pronounced varicosis with formation of odema, post traumatic swelling, after the healing of minor ulcers, after thrombophlebitis, to stabilise the results of sclerotherapy and varicotomy, more severe varicosis in pregnancy	mediven elegance (closed toe) mediven plus (open toe) mediven mondi ready to wear (closed and open toe) mediven for men (closed toe)
Class 3 For all late complications of constitutional or postthrombotic venous insufficiency, significant predisposition to odema, secondary varicosis, atrophie blanche, dermatosclerosis and after healing of severe ulcers, particulary recurrent ulcers	mediven plus (open toe)

Contraindications:

According to the guidelines of the German Phlebological Society (published 1996, German Journal, "Phlebolgie")

- **Absolute contraindications:** advanced peripheral arterial occlusive disease, decompensated congested heart disease, septic phlebitis, Phlegmasia coerulea dolens.
- **Relative contraindications:** supportive dermatoses, intolerance or compression stocking fabric, sensory disturbances of the limb, advanced peripheral neuropathy, primary chronic arthritis.

medi and its agents assume no liability as a result of any contraindicated use of this product