

Hospital Butler

The medi Hospital-Butler simplifies putting on thrombexin® thrombosis prophylaxis stockings and struva® anti-embolism stockings. The procedure is pain-free for the patient, and wound dressings do not slip. It also prevents damage to the stocking.



And this is how it's done:

1. Measure

Accurate measurements for the stocking ensure optimal medical care for the patient.




2. Select

The ankle circumference determines the size and thus selection of the suitable stocking. Adjust the width of the Hospital-Butlers so that it can pass over the patient's ankle and, if necessary calf, without touching the patient




3. Adjust the stocking

Place the stocking in the Butler and adjust the coloured marking at the heel until it is in the middle of the frame. Gather up the topband over the frame and pull the stocking down over the Butler. Grip the two central braces of the frame and pull the sides of the stocking alternately down over the frame until the colour markings are visible at the upper edge.

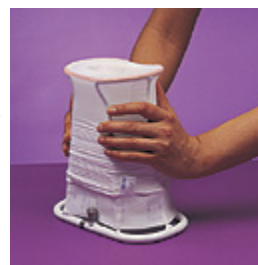

4. Hold the stocking

When pulling the stocking over the Butler hold it until the colour marking is at the level of the patient's heel.

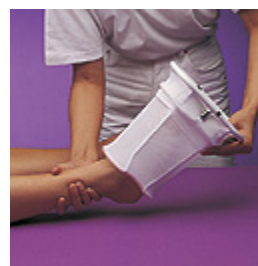

5. Push up the Hospital-Butler

Now release the stocking and push the Hospital-Butler up over the heel patient's and, if possible, over the knee.



6. Pull the Hospital-Butler downwards



7. Finishing touches

Now roll the stocking up over the knee and pull it towards the inguinal fold. The tension you exert determines the length of the stocking.



8. Inspection

Check that the stocking fits exactly and make sure it does not have any wrinkles.

